

ECG leads for custo cardio 1xx and custo guard 3

Electrode identifications

custo cardio 1xx

- R On the right below the collarbone
- L On the left below the collarbone
- F On the left above the hip

custo guard 3 (with custo belt 3)

- R On the right side of the chest
- L On the left side of the chest
- 3 On the right side of the back



ECG leads

custo cardio 1xx

- I = L - R
- II = F - R
- III = F - L

custo guard 3 (with custo belt 3)

- A = L - R
- B = 3 - R
- C = 3 - L

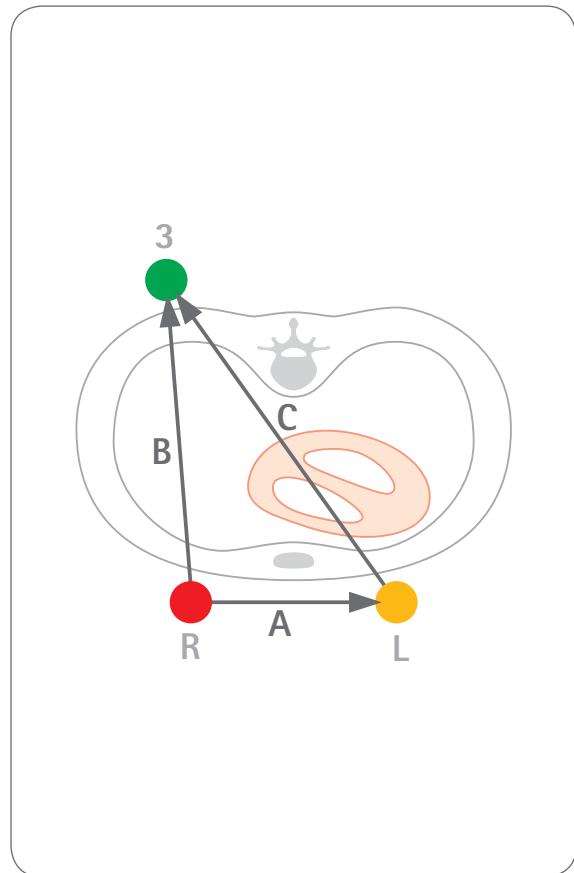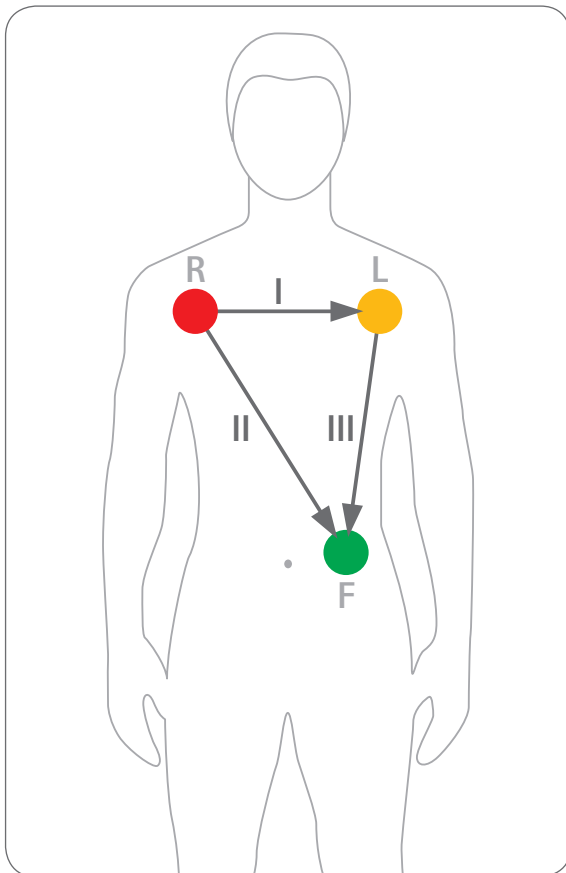

ECG leads for custo cardio 1xx and custo guard 3



Fig. 1:
Interference at electrode L ●
 with custo guard 3 and custo belt 3;
 causes interferences in leads A and C



Fig. 2:
Interference at electrode R ●
 with custo guard 3 and custo belt 3;
 causes interferences in leads A and B



Fig. 3:
Interference at electrode 3 ●
 with custo guard 3 and custo belt 3;
 causes interferences in leads B and C